

# **Elevate Digital Optics**

A Full-Service, Fully Independent Optical Lab

> (972) 488-3462 myedolab.com

- Every job crafted with care
- Latest and best lens technology
- Doctor-owned and operated
- Efficient and transparent service



# **PRODUCT GUIDE: TABLE OF CONTENTS**

Our Story	1
What Sets EDO Apart?	2
Materials	3
IOT Technology	4
Camber Steady Plus Progressive	5
Endless Steady Progressive	е
Essential Steady Progressive	7
Endless Office Occupational	8
Endless Progressive Options	9
Endless Bifocal	10
Endless Single Vision Options	11
Endless Drive	12
EDO Clear AR / Blue AR	13
Unity TechShield AR	14
IOT NeoChromes	15
Transitions Signature GEN 8	16
SunSync	17
NeuroLens Materials	18
NeuroLens Frame Verification Chart	19
NeuroLens Ordering Guide	20
Shamir Product Classification	21-22
Unity Options	23-25
Unity Fitting Guide	26
Unity Quick Reference Guide	27
Contact Us	28

# **OUR STORY**

# (972) 488-3462 myedolab.com



When we founded Elevate Digital Optics (EDO) in 2016, we knew exactly what we were up against. The market was (and still is) dominated by large, corporate labs that make you feel like a number instead of a valued customer. They take an impersonal approach, obscure their pricing as much as possible, and offer middle-of-the-road, commoditized products for the sake of their own interests. Even worse, the same companies run the industry's biggest insurance providers, requiring you to use their services and making it prohibitively difficult to take your business elsewhere.

At EDO, we set out to do things differently. We provide personalized service and speed in every order, and thanks to our technology partners, we promise only the best product selection in the industry, from our lens blanks to the designs and coatings we offer. We treat every customer as they deserve to be treated — we get to know you, your practice, and your patients. We pride ourselves on our transparency, from our system that allows you to track each order in real time through the steps of production, to our simple pricing, and our dedicated support team, standing by to help you manage any issues, changes, or last-minute requests that arise.

We even offer a way out from under the thumbs of the corporate labs with our shareholder program. When you work with EDO, you can choose to become a partner and retain an ownership stake in our business, freeing you to make decisions in the best interests of your patients and your practice.

Ultimately, we make a simple, yet critical promise, to everyone we serve: when you choose EDO, you can rest assured that we'll combine our knowledge, our people, our technology, and our commitment to the practices we serve to maintain an unmatched level of quality for every order we receive.

Sincerely,

Grant Guilbert | CEO
Peter Cass, OD | Board Chairman



## WHAT SETS EDO APART?

Elevate Digital Optics is dedicated to improving the way your practice manages optical orders. Here are a few things that make us different from the large, corporate labs to which we were founded as an alternative.

#### INDEPENDENT AND DOCTOR-OPERATED

We don't just want you to know us by our high-quality products — we want you to know us by our names. EDO started as a collective of independent eye doctors looking for a better alternative to a corporate lab. As we continue to grow, we vow never to forget where our business originated — and that extends to getting to know you and your practice on a personal level to guarantee top-notch service at every turn.

#### **EFFICIENCY AND TRANSPARENCY**

Because we're beholden to no one but our customers, we can do our best work faster and let you into every step along the way. In other words, there's no mysterious man behind the curtain with EDO. When you start sending jobs to us, you can track each and every order through the phases of production, ensuring that the final product will be as expected, every time. We're proud of our process and we love showing it off!

#### **EVERY JOB CRAFTED WITH CARE**

We're not a mass production facility and that's the way we like it. Treating every order as completely unique allows us to get the best results, because it means the eyewear we create is customized to your practice's requests and your patients' needs. The best end product doesn't come from simply duplicating the last job — we take pride in our work and our ability to give you and your patients the quality they deserve.

#### THE BEST TECHNOLOGY

As an independent lab, our production technology is a critical component to who we are and what we can accomplish. That's why we've chosen some of the most reputable partners to provide the materials, lens designs, and coatings as well as invested in the some of the latest and most trusted production equipment for our manufacturing floor. We strive to push our industry forward and never settle for less than the best.

#### **OUR SHAREHOLDER PROGRAM**

One of the aspects of EDO that our members like best is the opportunity to become more than a customer. After all, we're more than a vendor and we want our relationship with the practices we serve to be more than a transaction. As an EDO shareholder, you'll have agency in the decisions we make about the future of our business, take advantage of exclusive rebates, and get involved with a collaborative community of passionate ODs.

# MATERIALS OFFERED AT EDO LABS



Refractive index is the extent to which light is refracted when it enters a lens. The greater the index, the more refraction occurs – which means the slower the light moves through it. The slower the light moves, the more it bends. The more it bends, the less the thickness through which it needs to pass to achieve the same level of refraction, the result being a thin lens.

ABBE value is a measure of the amount of chromatic aberrations or refractive imperfections. The higher the ABBE value, the higher the overall clarity of the lens material.

All lenses at EDO (digital and conventional) come standard with scratch resistance and UV protection.

#### PLASTIC CR39 (INDEX1.49)

- Created as a lighter alternative to glass
- Absorbs tint/dye well
- Its density makes it easy to crack or shatter
- ☑ High ABBE value: 58



#### **TRIVEX (INDEX 1.53)**

- Most impact resistant material on the market
- Most resistant to fracturing, cracking or chipping in day-to-day handling
- ✓ Light weight: 8% lighter than polycarbonate; 16% lighter than CR-39; 19% lighter than high-index
- ABBE Value: 45



#### **POLYCARBONATE (INDEX 1.59)**

- About 20% thinner than plastic or glass lenses
- 10 x's more impact resistant than CR39
- 100% UV protection
- ✓ ABBE Value: 30



#### **HIGH INDEX 1.60**

- Best clear alternative to poly and CR39
- About 20-25% thinner than CR39 plastic
- Absorbs tint/dye well
- ABBE Value: 42



#### **HIGH INDEX 1.67**

- Best clarity and thinness for high prescriptions
- Up to 35% thinner than CR39 plastic
- Aspheric design reduces curvature, which reduces magnification and improves appearance
- ABBE value: 32



#### **HIGH INDEX 1.74**

- Thinnest material on the market
- ☑ Up to 50% thinner than CR39 plastic
- Aspheric design reduces curvature, which reduces magnification and improves appearance
- Requires premium coatings, scratch protection and/or AR
- ABBE Value: 33



#### Lens Optimization with IOT Digital Ray-Path 2 Technology

Single vision lens, [+3.00-1.00 x 90], 6 D base curve and 1.5 in dex

99.5% of any direction of gaze is fully optimized when the wearer accommodates slightly. IOT Digital Ray-Path 2 lenses have virtually no full-field blur in any gaze direction.



IOT DIGITAL RAY-PATH 2 TECHNOLOGY



IOT Digital Ray-Path 2 harnesses the intrinsic potential of the visual system to refine the optimization process for personalized lenses. It analyzes oblique aberrations at various focal distances for each direction of gaze. Minimization of oblique aberrations is balanced throughout object space, providing extremely clear vision and precise focus.

Camber Technology combines complex surfaces on both sides of the lens to provide excellent visual correction. Its unique front surface has an unique variable base curve. It is specially designed to increase the reading areas and achieve better optical quality.



# STEADY METHODOLOGY AND STEADY PLUS METHODOLOGY

**STEADY METHODOLOGY** is a technological breakthrough in free-form, digital lenses. In Addition to controlling for unwanted cylinder power, Steady Methodology addresses unwanted changes to mean power in the lateral areas of the lens. This improves peripheral visual acuity, reduces swim effect, and provides superior image stability and offers more comfortable vision.

**STEADY PLUS METHODOLOGY** represents an evolution of Steady Methodology. It carefully balances the needed sphere power to achieve a perfectly symmetrical and smooth distribution on both sides of the lens.

Distribution of spherical equivalent with Steady Methodology and Steady Plus Methodology Rx: Plano Sph. +2.00







Traditional

Lens with Steady Methodology

Lens with Steady

PORTFOLIO		TECHNOLOGIES	LIFESTYLE	MFHs
	Camber Steady Plus Progressive	CHAMBER DRP 2 STEADY +	Near, distance, and intermediate enhancements	14, 15, 16, 17, 18
7.2	Endless Steady Progressive	DRP 2 STEADY	Near, distance, and intermediate enhancements	14, 15, 16, 17, 18
	Essential Steady Progressive	STEADY	Near, distance, and intermediate enhancements	14, 15, 16, 17, 18
4	Endless Office Occupational	DRP 2	Focal lenghts: 1.3 m   2 m   4m	14, 18
	Endless Bifocal	DRP 2	Segment diameters: 32 mm   40 mm	14
4	Endless Sport Progressive	DRP 2	Panoramic distance vision for outdoor activities	16, 18
	Endless Pilot Progressive	DRP 2	Extra segment for near vision at the top of the lens	16, 18
	Endless Drive Progressive and Single Vision	DRP 2	Perfect for driving in all lighting conditions - night vision zone	PAL: 16, 18
	Endless Anti-fatigue Single Vision	DRP 2	Power boosts: 0.50 D   0.75 D   1.00 D	16, 18
	Endless Single Vision	DRP 2	Unparalleled visual quality	-

# CAMBER STEADY PLUS PROGRESSIVE

#### **Benefits**

- Superior visual acuity.
- ▶ Improved quality of vision in the near zone.
- ▶ Improved aesthetics in many prescriptions.
- Precise and comfortable focus for all working distances in any direction of gaze.
- ▶ Near elimination of peripheral blur.
- Superior visual quality for viewing digital
- ▶ Higher image stability for reduced swim effect.

## **Performance** Distance Near Intermediate **Technologies** Personalized CAMBER STEADY +

# **OPTIONS**

#### **CAMBER STEADY PLUS PROGRESSIVE**



#### **NEAR VISION**


**IDEAL WEARER:** expert or novice progressive lens wearers who have visual demands that call for a larger more usable near area

#### **PERFORMANCE**

Distance Near



#### **CAMBER STEADY PLUS PROGRESSIVE**

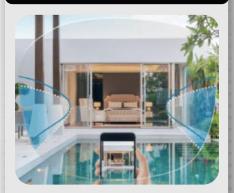

#### **DISTANCE VISION**

**IDEAL WEARER:** expert or novice progressive lens wearers who have visual demands that call for a larger more usable distance area.

#### **PERFORMANCE**


Distance

Near

Intermediate



#### **CAMBER STEADY PLUS PROGRESSIVE**

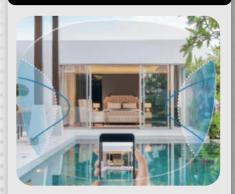

#### **INTERMEDIATE VISION**

**IDEAL WEARER:** first-time wearers, wearers who have experienced non-adapts with other progressive lenses, and those who have visual demands that call for a larger more usable intermediate area.

#### **PERFORMANCE**

Distance

Near



# ENDLESS STEADY PROGRESSIVE

#### **Benefits**

- ▶ Precise and comfortable focusing for all working distances in any direction of gaze.
- ▶ Near elimination of peripheral blur.
- Superior visual quality when using digital devices.
- ▶ Higher image stability for reduced swim effect.
- ▶ Improvement of peripheral visual acuity in the distance zone.



# **OPTIONS**

# ENDLESS STEADY PROGRESSIVE



#### **NEAR VISION**

**IDEAL WEARER:** expert or novice progressive lens wearers who have visual demands that call for a larger more usable near area

#### **PERFORMANCE**

Distance Near

Intermediate



# ENDLESS STEADY PROGRESSIVE



#### **DISTANCE VISION**

**IDEAL WEARER:** expert or novice progressive lens wearers who have visual demands that call for a larger more usable distance area.

#### **PERFORMANCE**

Distance

Near

Intermediate



# ENDLESS STEADY PROGRESSIVE



#### INTERMEDIATE VISION

**IDEAL WEARER:** first-time wearers, wearers who have experienced non-adapts with other progressive lenses, and those who have visual demands that call for a larger more usable intermediate area.

#### **PERFORMANCE**

Distance

Near



# ESSENTIAL STEADY PROGRESSIVE

#### **Benefits**

- ► Higher image stability for reduced swim effect
- ▶ Improvement of peripheral visual acuity in the distance zone.

# Performance

**Technologies** 

STEADY

Distance Near Intermediate



Non-Personalized

# **OPTIONS**

# ESSENTIAL STEADY PROGRESSIVE



#### | NEAR VISION

**IDEAL WEARER:** expert or novice progressive lens wearers who have a need for a high value lens and visual demands that call for a larger more usable near area.

#### **PERFORMANCE**

Distance Near Intermediate



# ESSENTIAL STEADY PROGRESSIVE



#### **DISTANCE VISION**

**IDEAL WEARER:** expert or novice progressive lens wearers who have a need for a high value lens and visual demands that call for a larger more usable distance area.

#### PERFORMANCE

Distance Near Intermediate



# ESSENTIAL STEADY PROGRESSIVE



## | INTERMEDIATE VISION

**IDEAL WEARER:** first-time wearers or wearers who have experienced non-adapts with other progressive lenses. Those who have a need for a high value lens and visual demands that call for a larger more usable intermediate area.

#### **PERFORMANCE**

Distance Near



# ENDLESS OFFICE OCCUPATIONAL

#### **Benefits**

- ▶ Maximum intermediate and near visual fields.
- ▶ Improved postural ergonomics avoiding unnecessary head movements.
- Comfortable and precise focusing, especially when using electronic devices.
- Excellent dynamic vision, easy transition between near and intermediate visual fields.
- ▶ Immediate adaptation.
- ▶ Near elimination of peripheral blur.
- ▶ Superior visual quality when using digital devices.

Technologies

Personalized

DRP 2

## **OPTIONS**

# ENDLESS OFFICE OCCUPATIONAL | 1.3 M



#### CLEAR VISION from 35 cm (14 in) to 1.3 m (4.2 ft)

**TYPES OF USE:** when clear vision is required at desk level, prolonged use at very near distances.

**IDEAL WEARER:** Presbyopes who work in a small space and spend a significant amount of time focusing on very near distance. For Example, office workers who spend most of their time viewing monitors on a desk.

#### **PERFORMANCE**

Distance Near

Intermediate



# ENDLESS OFFICE OCCUPATIONAL | 2 M



#### CLEAR VISION from 35 cm (14 in) to 2 m (6.5 ft)

**TYPES OF USE:** when clear vision is required at desk level, prolonged use at intermediate and near distances.

**IDEAL WEARER:** Presbyopes who work in a roomsized space and spend a significant amount of time focusing at intermediate and near distances. For example, office workers who utilize monitors but have a need to view another person at a conversational distance.

#### **PERFORMANCE**

Distance

Near

Intermediate



# ENDLESS OFFICE OCCUPATIONAL | 4 M



#### CLEAR VISION from 35 cm (14 in) to 4 m (13.1 ft)

**TYPES OF USE:** when clear vision is required in a room-sized space and extra viewing is needed at intermediate and near distances.

**IDEAL WEARER:** Presbyopes who work in a larger space and spend time focusing on intermediate and near distances. For example, people who utilize monitors and have a need to view another person at a conversational distance and move about their workspace.

#### **PERFORMANCE**

Distance

Nea



# ENDLESS DRIVE PROGRESSIVE




#### | BENEFITS

- ▶ Improves the visual experience of the wearer when driving in daytime and nighttime conditions.
- ► Compensates for the effects of night myopia with a unique zone to provide better focus.
- ▶ Optimized vision for a better view of the dashboard and mirrors.
- ▶ Reduces visual fatigue symptoms when driving at night.
- Greater visual acuity for easy focus and more agile eye movement.
- ▶ Near elimination of peripheral blur.

#### **PERFORMANCE**

Distance Near Intermediate

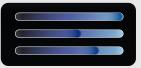

**Technologies** 

Personalized

DRP 2

# ENDLESS SPORT PROGRESSIVE



#### BENEFITS

- Maximum distance vision
- Unmatched dynamic vision with comfortable and precise focus at any distance.
- Optimized fields of view, even with highly wrapped frames and higher prescriptions.
- ▶ Near elimination of peripheral blur.

#### **PERFORMANCE**

Distance
Near
Intermediate



**Technologies** 

Personalized

DRP 2

# ENDLESS PILOT PROGRESSIVE



#### **BENEFITS**

- Precise and comfortable near vision through the upper and lower areas of the lens.
- ▶ Improved postural ergonomics avoiding unnecessary head movements.
- ► Comfortable and precise focus at all working distances.
- ► Excellent dynamic vision, easy transition between different viewing areas.
- ▶ Upper segment adapted to the wearer's visual needs.
- ▶ Near elimination of peripheral blur.

#### **PERFORMANCE**

Distance Near Intermediate



Technologies

Personalized

DRP 2

# **ENDLESS BIFOCAL**

#### **Benefits**

- Wide areas of view at near and distance, free of aberrations.
- ▶ Better aesthetics, less visible segment line.
- ▶ Easy transition between visual fields.
- Comfortable and precise focusing, especially when using electronic devices.
- ▶ Countless material and treatment options.
- ▶ High value solution.
- ▶ Near elimination of peripheral blur.

Technologies

Personalized

DRP 2

# **OPTIONS**

#### **ENDLESS BIFOCAL**



#### ROUND

The diameter (32 mm) and position of the segment are very similar to a conventional bifocal with straight, curved, or round segments. This solution is perfect for those who already wear bifocal lenses and want easy adaptation to their new eyeglasses.

#### **PERFORMANCE**

Distance Near



#### **ENDLESS BIFOCAL**



#### | WIDE

In the configuration, the segment is expanded up to 40 mm, offering a noticeably larger near viewing area while maintaining panoramic distance vision. The configuration is ideal for current wearers of wide segment conventional or executive bifocals. It is also optimal for vision therapy for children with accommodative difficulties.

#### **PERFORMANCE**

Distance
Near

#### ENDLESS SINGLE VISION



#### | BENEFITS

- ▶ Impeccable visual quality, especially for high prescriptions and wrapped frames.
- ► Comfortable and accurate focusing at all distances.
- ► Superior visual quality for viewing digital devices.
- ▶ Near elimination of peripheral blur.

# ENDLESS ANTI-FATIGUE SINGLE VISION



#### | BENEFITS

- ▶ Impeccable visual quality and precise focus.
- ► More relaxed vision with less accommodative effort
- ▶ Designed to significantly emproved reading speed on digital devices.
- ► Comfortable and precise focus at all distances.
- ► Superior visual quality for viewing digital devices.
- Near elimination of peripheral blur.

Personalized

# ENDLESS DRIVE SINGLE VISION



#### | BENEFITS

- Improves the visual experience of the wearer when driving in daytime and nighttime conditions.
- ► Compensates to the effects of night myopia with a unique zone to provide better focus.
- Optimized vision for a better view of the dashboard and mirrors.
- ▶ Reduces visual fatigue symptoms when driving at night.
- ► Greater visual acuity for easy focus and more agile eye movement.
- Clear vision from center to edge ensuring a comfortable lens with no limitations regardless of the prescription or frame selected.
- ▶ Near elimination of peripheral blur.

Technologies

Personalized

DRP 2

Technologies

Personalized

DRP 2

DRP 2

**Technologies** 

# Ot

# ENDLESS DRIVE SINGLE VISION

# SINGLE VISION

# Night vision zone Compensates for the effects of night myopia up to 0.25D. The power of the lens is compensated. The design is calculated, point-by-point, to ensure wearers perceive the proper power when looking through their lenses at every distance and direction of gaze.

## **FEATURES**



Personalization



Compensated Power




Optimized for Accomodative Object Space



Consistency

# **BENEFITS**

- ▶ Improves the visual experience of the wearer when driving in daytime and nighttime conditions.
- ► Compensates for the effects of night myopia with a unique zone to provide better focus.
- ► Reduces visual fatigue symptoms when driving at night.
- ► Greater visual acuity for easy focus and more agile eye movement.
- Optimized vision for a better view of the dashboard and mirrors.

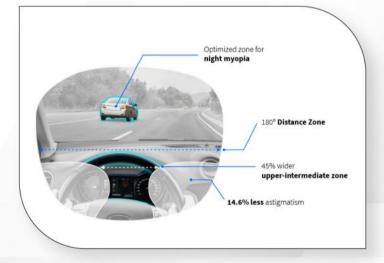

# ICI ENDLESS DRIVE PROGRESSIVE

## **FEATURES**

# **BENEFITS**

- ▶ Provides exceptional visual acuity at all distances.
- ▶ Improves the visual experience of the wearer when driving in daytime and nighttime conditions.
- Compensates for the effects of night myopia with a unique zone to provide better focus.
- Reduces visual fatigue symptoms when driving at night.
- ▶ Greater visual acuity for easy focus and more agile eye movement.
- ▶ Optimized vision for a better view of the dashboard and mirrors.

Personalization



Compensated Power



Optimized for Accomodative Object Space




Consistency



# **EDO Clear AR**

## LIGHT ENHANCING TECHNOLOGY A R



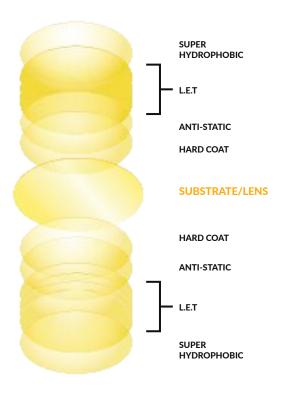



Only UVAR available in multiple colors



Super hydrophobic coating



Reduces UV backside reflection



includes anti-static layer



## THE DIGITAL EYE STRAIN SOLUTION!

Blocking the irritating effects of electronics give the eyes rest from daily exposure.

**EDO Blue AR makes the digital life less of a strain!** 



#### **BLUE LIGHT ABSORPTION**

Filters harmful blue-violet HEV Light (up to 17%) in the 400-455nm range, which are more destructive, shorter wavelengths.



#### VISIBLE LIGHT **TRANSMISISON**

**Provides excellent Light** Transmission in the visible range, allowing beneficial light to pass through & provide great visual acuity.



#### **BACKSIDE HEV ABSORPTION**

Includes a proprietary backside coating which transmits blue light away from the eyes and reduces backside reflection to <1%



#### **DIGITAL EYE STRAIN RELIEF**


Reduces: eye strain, headaches, blurred vision, dry eyes, and neck, shoulder & back pain associated with blue-violet **High Energy Visible** (HEV) light and the use of electronic devices, fluorescent lighting, flat screens and LED's.



**REFLECTION** 



**ABSORPTION** 

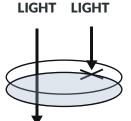

**HEV** 

**GOOD** 



**BACKSIDE UV** REFLECTION



**BACKSIDE UV** 

# techshield ANTI-REFLECTIVE COATINGS



UNCOATED LENS









TECHSHIELD AR COATED LENS









#### **TECHSHIELD ELITE**

#### ULTIMATE VISUAL PERFORMANCE, COSMETIC APPEARANCE, DURABILITY, AND CLEANABILITY.

- 2x Better Cleanability [1]
- 2x Better Scratch Resistance [2]
- Excellent Visual Clarity
- Exceptional Visual Clarity
- Attractive Cosmetic Appearance
- UV Protection (Optional UVR)
- 2-year Warranty [3]

#### **TECHSHIELD PLUS**

## SUPERIOR VISUAL PERFORMANCE AND AESTHETICS WITH ADDED DURABILITY

- 2x Better Cleanability [1]
- 2x Better Scratch Resistance [2]
- Excellent Visual Clarity
- Great Smudge Resistance
- Attractive Cosmetic Appearance
- UV Protection (Optional UVR)
- 2-year Warranty [3]

#### **TECHSHIELD**


## EXCEPTIONAL VISUAL PERFORMANCE IN AN ATTRACTIVE BASE ENHANCEMENT.

- 2x Better Cleanability [1]
- 2x Better Scratch Resistance [2]
- Excellent Visual Clarity
- Good Smudge Resistance
- Attractive Cosmetic Appearance
- UV Protection (Optional UVR)
- 2-year Warranty [3]

#### **TECHSHIELD BLUE**

## TARGETED BLUE LIGHT DEFENSE IN A NEXT-GENERATION AR COATING.

- Targeted Blue Light Reduction
- 2x Better Cleanability [1]
- 2x Better Scratch Resistance [2]
- Excellent Visual Clarity
- Excellent Smudge Resistance
- Attractive Cosmetic Appearance
- UV Protection (Optional UVR)
- 2-year Warranty [3]

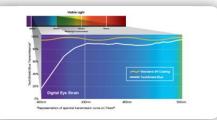


TechShield Blue targets blue light wavelengths associated with digital eye strain (400-430nm), reducing exposure by as much as 85 % at its peak.

- **1.** Compared previoys versions of TechShield AR Coatings in laboratory testing.
- **2.** Compared to previous versions of TechShield AR Coatings in laboratory testing.
- **3.** TechShield Anti-Reflective Coatings are backed by a two-year warranty againts scratchingm peeling, and crazing under normal use. Lenses that are abused, extremely scratched, chipped, brolen or damaged are not covered under this warranty.

#### PRACTICE RESOURCES:

www.techshieldar.com/rsources



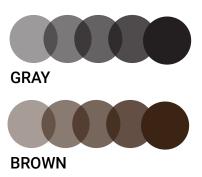

- Quickly adapts to changing light conditions
- 3 minute fade-back\*
- Block 100% of damaging UVA and UVB rays
- Provides high protection against blue light when dark
- Consistent performance across a wide range of materials
- **High performance** in a broad range of temperatures

Material	Color	Diameter	Base curve
Material	Gray & Brown	76	1.25   2.25   3.25   4.25   5.25   6.25   7.25   8.25
Trivex*	Gray & Brown	70 75	Diameter 70: 1.00   Diameter 75: 2.50   4.00   5.00   6.00   7.00   8.00
Polycarbonate*	Gray & Brown	76	0.50   1.25   2.25   3.25   4.25   5.25   6.25   7.25   8.25
Hi-index 1.60	Gray & Brown	73	0.50   1.00   2.00   3.00   4.00   5.00   6.00   7.00   8.00
Hi-index 1.67	Gray & Brown	75	1.00   2.50   4.00   5.00   6.00   7.00   8.00

# NEOCHROMES® PARADISE COLLECTION

**Day or night, be yourself with Paradise Collection** mirrored photochromic lenses.

Paradise Collection Light Activated Lenses come in 3 distinctive colors designated to make your patient's lenses look great in both their light and dark states.

MATERIAL	PHOTOCHROMIC COLOR	MIRROR COATING	DIAMETER	BASE CURVE	RANGE
Polycarbonate	Gray	Fire, Ocean, Orchid	76	4.25/6.25/8.25	+8.00 to 6.00





#### Transitions® Signature® GEN 8™

Suitable for any age, including children. Transitions<sup>®</sup> Signature<sup>®</sup> GEN 8<sup>™</sup> lenses are ideal for eyeglasses wearers who are both inside and outside during the day, desire the clearest lens indoors, are new to photochromic lenses or are seeking the latest style trends.



- ▶ Transitions® most responsive lens
- ▶ Exclusive GEN 8™ technology
- ▶ Superior visual experience by being more reactive to every type of light, from sun to indirect



- Completely clear indoors and dark outdoors
- Intelligently adjusts to the perfect tint whatever you rlight
- Consistently performs in all temperatures



- Helps protect againts harmful blue light - Indoors 20% / Outdoors 87%+
- Blocks 100% of UV rays



The #1 availability in material, design and colors to style any frame







+4



Transitions® Signature® GEN 8™ Style Colors

The best-selling photochromic lens is now available in an enhanced choice of stunning lens colors to personalize any frame and express any style









#### XTRACTIVE® NEW GENERATION



in Hot Temperatures



LIVA & LIVB



The Darkest in Car



Best Blue Light Protective Indoors

3 COLORS



Clear Indoors with a Hint of Protective Tint



Up to 35 % Faster Fadeback

#### XTRACTIVE® POLARIZED™



Up to 90% Polarization Efficiency



Extra-dark up to Category 36



Activates in the Car



Block 100% LIVA & LIVB




Best Blue Light Protective Indoors



Up to 2x Faster Fadeback<sup>8</sup>

COLOR

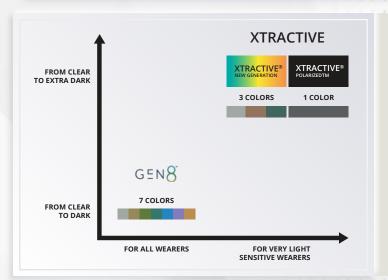

For those who are very light sensitive and/or frequently exposed to very bright lights, the new Transitions XTRActive range of extra dark lenses - led by Transitions XTRActive **new generation** - are uniquely designed to meet their need for the **best extra darkness and blue light** protection9.

Now this incredible range is also introducing *Transitions*® XTRActive® Polarized™ lenses - the only and best ever photochtomic polarized lens<sup>12</sup> that activates from clear indoors to dark and polarized ourdoors.

9 Transitions XTRActive new generation; the darkest in hot temperatures and in the car, achieving maximum darkness (category 3 levels) @2332C and offering the best overall blue light protection across light situations among clear to extra dark Photochromic lenses.

12 Compared to clear to extra dark photochromic lenses.



LIGHT-REACTIVE LENSES

ULTRA	-FAST	CONVENTIONAL			
SunSyc Elite	SunSync Elite XT	SunSyc Elite	SunSync Elite XT		
Ultimate lens for patients seeking fastest fade-back and clear appearance indoors, but don't need darkest color outdoors or in-car color retention.	Perfect for patients who want an extra-dark light-reactive lens with fast fade-back, excellent UV and blue light defense, and in-car color retention.	All-purpose lens for patients seeking solid performance across the board, but don't need the fastest fade-back speed, darkest color, or in-car color retention.	Great daily lens for patients looking for extra-dark outdoor color, in-car activation, and exceptionally clear indoor appearance, but don't need the fastest fade-back speed.		
Recommend For:  • Fastest fade-back speed  • Clear indoor appearance  • UV and blue light defense  • Exceptional durability	Recommend for:  Ultra-fast fade-back speed Extra-dark outdoor color UV and blue light defense In-car color retention	Recommend for:  Dark outdoor color  Clear indoor appearance  UV and blue light defense  Exceptional durability	Recommend for: In-car color retention Darkest outdoor color UV and blue light defense Clear indoor appearance		
Lifestyle Fits: Any lifestyle benefitting from the fastest fade back, clearest indoor appearance, UV and blue light, defense, and dark, but not the darkest color.	Lifestyle Fits: Outdoor lifestyles or those with regular indoor-outdoor movement requiring the darkest color, UV and blue light defense, and fast, but not the fastest fade back.	Lifestyle Fits: Busy lifestyles with indoor-outdoor movement that don't require fastest reaction speed, darkest outdoor color, or in-car activation.	<b>Lifestyle Fits:</b> Lifestyle that requires ability for activation in all environments, including the car.		
Occupational Fits: Occupations with frequent indoor-outdoor movement or longer periods indoors, need for clearest appearance, and regular screen, or device use.	Occupational Fits: Outdoor occupations, driving professions with regular movement in and out of car, and jobs with regular screen or device use.	Occupational Fits: Occupations that entail longer periods indoors with intermittent trips outside.	Occupational Fits: Outdoor occupations, driving occupations, indoor occupations with extended periods outdoors.		

**Traditional Photochromic** 

(Responds to UV light only)



**SunSync Drive XT** 


(Responds to UV and visible light)





Incorporating contoured prism technology. Neurolenses bring the eyes back into alignment, providing symptom relief for patients in addition to sharp, comfortable vision.

Available in Single Vision, Office and Progressive designs, the proprietary contoured prism in Neurolenses increases 0.375 diopters in each lens (0.75 diopters total) from distance to near.

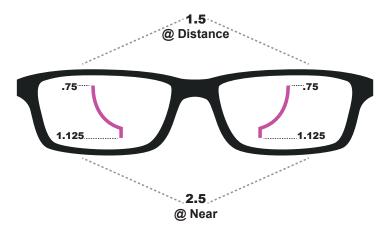

Standard vertical prism can be incorporated into the Neurolens. Minimum fitting height is 18mm.

#### Lens Material

- 1.53 Trivex
- 1.60 High Index
- 1.67 High Index

Blue Block lens material reduces harmful HEV rays vs standard lenses





Neurolens SV



Neurolens PAL
Smooth power
progression



30% power additional at fitting point calculated automatically

## Coatings

**Premium** is an enhanced AR coating with backside UV protection, providing smudge and water resistance, dust and scratch protection and reduced reflections.


**Premium+** is superior AR coating with backside UV protection that repels water, oils, dust and smudges, and eliminate reflections for improved vision, with durable scratch resistance.


**NeuroBlue** is superior AR coating that offers all the benefits of Premium+ and include additional protection against harmful blue light.






#### Frame Verification Chart

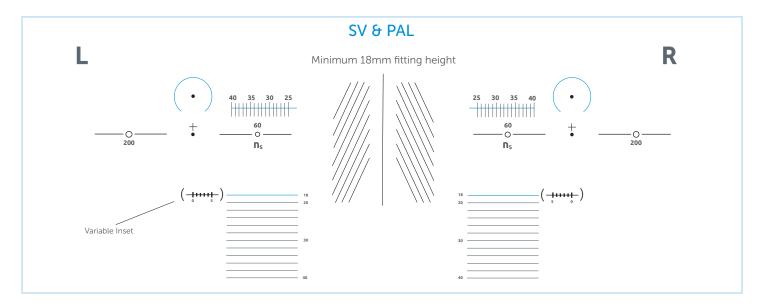

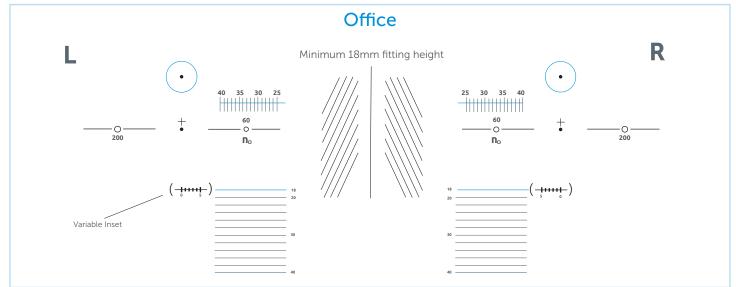





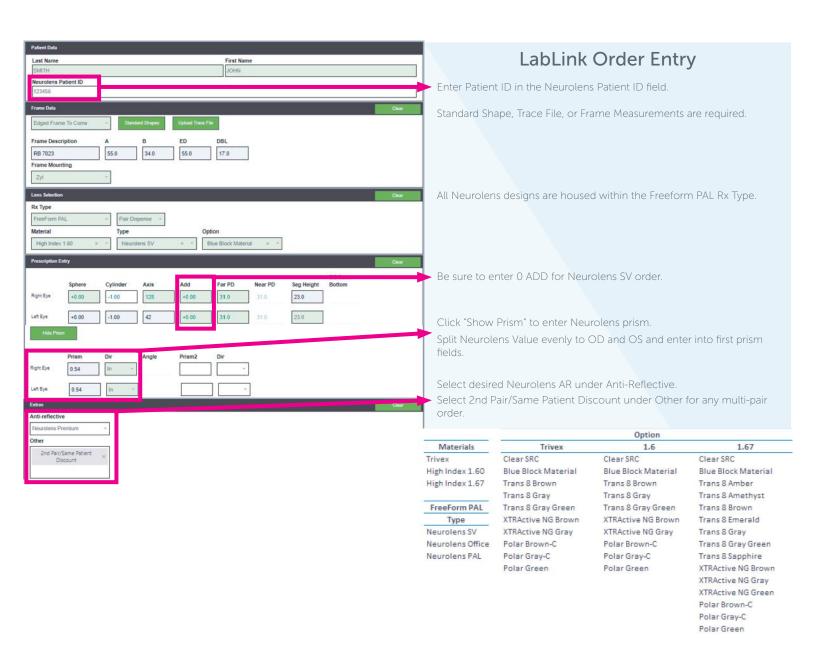





PROGRESSIVE	VISUAL SOLUTION	MIN FITTING HEIGHT
Autograph Intelligence* **		11mm, 13mm, 15mm, 18mm & Variak
Autograph® III **		11mm, 13mm, 15mm, 18mm & Varial
Autograph® II+*		11mm, 13mm, 15mm, 18mm & Varial
InTouch"	Progressive	15mm & 18mm
Spectrum+ <sup>™</sup>		14mm, 16mm & 18mm
Element <sup>™</sup>		16mm & 19mm
FirstPAL"		15mm
Genesis HD <sup>™</sup>		16mm
BIFOCAL	VISUAL SOLUTION	MIN FITTING HEIGHT
Duo <sup>™</sup>	Bifocal	15mm
SINGLE VISION	VISUAL SOLUTION	MIN FITTING HEIGHT
Autograph® III - SV **		Providing a fitting height is recommend Please see White Paper. <sup>1</sup>
Autograph II - Single Vision**	Single Vision	Providing a fitting height is recommend Please see White Paper. <sup>1</sup>
Relax 50•65•80	33.	14 mm
Spectrum - Single Vision <sup>™</sup>		Providing a fitting height is recommend Please see White Paper. <sup>1</sup>
PLAY	VISUAL SOLUTION	MIN FITTING HEIGHT
Attitude® III - Fashion **	Progressive	15mm & 18mm
> SPORT	VISUAL SOLUTION	MIN FITTING HEIGHT
Attitude® III - Sport **		18mm
Golf <sup>™</sup> *	Progressive	19mm
Autograph II - Attitude <sup>™</sup> *		15mm & 18mm
SPORT SINGLE VISION	VISUAL SOLUTION	MIN FITTING HEIGHT
Attitude® III - SV **		
Golf - Single Vision" *	Single Vision	Providing a fitting height is recommend Please see White Paper. <sup>1</sup>
Autograph II - Single Vision Attitude <sup>™</sup> *		
WORK	VISUAL SOLUTION	MIN FITTING HEIGHT
Computer <sup>**</sup>		14mm
	Occupational	

<sup>\*\*</sup> Utilizes As-Worn Quadro". See product flyer for more information.

White Paper can be accessed at shamirlens.com under the *Professionals* section.



<sup>\*</sup> Utilizes As-Worn Technology". See product flyer for more information.

#### Shamir Autograph Intelligence™\*\*

Shamir Autograph Intelligence" is a new progressive lens whose vision zones are designed according to the visual needs of the patient's Visual Age". This design incorporates all of Shamir's tried and true technologies, as well as the enhanced Eye-Point Technology AI\*, Continuous Design Technology\*, and Shamir's Visual AI Engine<sup>™</sup>.

#### Shamir Autograph® III \*\*

Shamir Autograph® III is a balanced progressive lens, holistically adapted to the patient's prescription, ensuring a clear and comfortable visual experience for all! Utilizing the most advanced features and technologies: EyePoint Technology III®, Natural Posture<sup>™</sup>, IntelliCorridor<sup>™</sup> and As-Worn Quadro<sup>™</sup>, Autograph<sup>®</sup> III represents the most personalized progressive solution on the market.

#### Shamir Autograph® II+ \*\*

With the addition of Natural Posture" and Close-Up", Shamir Autograph  $^{\circ}$  II+ provides patients more comfort both ergonomically and visually. Natural Posture ensures a comfortable ergonomic posture while reading, and Close-Up™ enables a more natural convergence of the eyes while doing close tasks.

#### Shamir InTouch"

An everyday progressive lens solution with enhanced visual comfort in the zone most utilized for handheld technology, patients will notice a quicker transition of add power in the intermediate zone (up to 25% quicker than standard progressives). InTouch<sup>™</sup> provides the visual comfort patients need for the "handheld zone," which is between 15" - 27".

#### Shamir **Spectrum+**™

Shamir Spectrum+<sup>™</sup> is more than a standard everyday progressive solution. Featuring Natural Posture", patients now reap the benefits of a more ergonomic posture while reading. It means less neck aches, and avoiding having to hold reading material in uncomfortable positions.

#### Shamir **Element**™

Let's face it, consumers want high quality products at a reasonable price, we understand that... after all we're consumers too. With Shamir Element" (we like to call it the "elementary Freeform® lens") your patients get a personalized backsurfaced lens, that incorporates EyePoint Technology®, Direct Lens Technology® and is created with a Freeform® machine, so the Rx and design are on the back! It's simply frugaltastic!

#### Shamir FirstPAL™

Shamir FirstPAL™ features wider reading and intermediate zones and the full add is obtained quickly, so they won't have to lower their heads as much to see, like they would in a standard progressive.

#### Shamir Genesis HD

Shamir Genesis HD introduces patients to a budget-friendly Freeform® technology solution that offers a semi-personalized lens option for any prescription.

#### **EVERYDAY> Single Vision**

#### Shamir Autograph® III - SV \*\*

Autograph® III - SV is the most personalized digital lens design your single vision patients will experience today. Autograph® III - SV incorporates the most revolutionary technologies (EyePoint Technology III® and As-Worn Quadro") for the widest possible clear vision, further enhanced by accommodating the way your patient wears their frame.

#### Shamir Spectrum - Single Vision™

Shamir Spectrum - Single Vision" introduces patients to the world of Freeform®. While we do offer a fully personalized single vision lens in the Autograph II -Single Vision\*, some patients just aren't ready to take the leap into something so revolutionary... Spectrum – Single  $\mathsf{Vision}^{\scriptscriptstyle{\mathsf{T}}}$  provides patients a technologically advanced single vision lens that delivers a precise Rx at a price that fits any budget.

#### Shamir Relax™

Shamir Relax® eases the strain normally associated with prolonged accommodation. How can a lens leave your patients more energized? By ADDing a little extra power into the lens, Relax\* helps reduce accommodation by up to 30%! Trust us, your patients are going to love you when you put them in Relax".

#### **EVERYDAY> Bifocal**

#### Shamir **Duo**™

This new generation of Freeform® bifocal lens eliminates the visible dividing line associated with traditional bifocals. The smooth surface means no image jump or annoying line along the natural eye path, and allows for a wide choice of materials, treatments and premium coatings.

#### **PLAY**

#### Shamir Attitude® III - Fashion \*\*

Designed to provide expanded vision zones and enhanced peripheral vision, this progressive sun lens fits in even the biggest fashion sunglass frames. With balanced far, intermediate and near vision, Attitude® III - Fashion is perfect for any day-to-day outdoor activities.

#### PLAY> Sport

#### Shamir Attitude® III - Sport \*\*

Suited specifically for sports and active lifestyles, Shamir Attitude® III - Sport's vision zones are perfectly adapted for dynamic sports, featuring optimal vision in the Safety Zone as well as a wide and distortion-free far vision zone.

#### Shamir Golf<sup>™</sup>\*

Golfers don't always need to blame high scores on a bad day. Vision can be a driving force behind an above par game. Unlike general purpose progressives, Shamir Golf" is optimized for the golf course. Golf" is designed to meet the visual needs of the golfer by focusing on three crucial zones: score card, ball on the tee, and green in the distance. Now a shoddy day on the course can be a hole-in-one day!

#### PLAY> Sport Single Vision

#### Shamir Attitude® III - SV \*\*

Designed to provide single vision wearers with maximized visual comfort for all of their sport and outdoor activities and with greater design stability in any frame large or small, wraparound or flat - Attitude® III - SV provides expanded peripheral vision unlike any single vision sun lens.

#### Shamir Golf - Single Vision" \*

For those single vision patients that take part in the relaxing game of golf, we've developed Shamir Golf - Single Vision\*. It's a single vision lens designed to provide clear, crisp vision for wrap frames... The perfect accessory for any golfer!

#### WORK

#### Shamir Computer™

When focusing on a computer and its near surroundings is a top priority, Shamir Computer<sup>™</sup>, an advanced Freeform<sup>®</sup> lens, provides a wide field of near viewing with clear vision up to 5ft. - a far greater depth of field than that offered by conventional reading lenses.

#### Shamir WorkSpace™

Shamir WorkSpace" is the best solution when priority focus is both mid-distance and near viewing. Offering a greater depth of field, up to 10ft., this advanced and highly sophisticated Freeform® lens design, allows movement within the workspace and sharp vision whether the required focus is on the desk or a colleague across the room

- \*\* Utilizes As-Worn Quadro". See product flyer for more information.

  \* Utilizes As-Worn Technology". See product flyer for more information

#### Technical Inquiries: 888.370.0736







/ShamirInsight



@Shamir\_Insight



thevitaminsee.com

		CR-39	Polycarbonate	Trivex	1.60	1.67	1.74
N	•	•	•	•	•	•	•
0	•	•	•	•	•	•	•
0	•	•	•	•	•	•	•
F		•	•	•	•	•	•
0	•	•	•	•	•	•	
		All	All	All		All	
	O O F	0 • • • • • • • • • • • • • • • • • • •	O • • • • • • • • • • • • • • • • • • •	O • • • • • • • • • • • • • • • • • • •	O	O	O

Patient Benefits: Clear vision near, far, and everywhere in-between with a single pair of glasses.

Visual Need: Presbyopres requiring multifocal lenses.

Design	VSO® Code	CR-39	Polycarbonate	Trivex	1.60	1.67
Unity Via OfficePro 5 FT.	N	•	•	•	•	•
Unity Via OfficePro 10 FT.	0	•	•		•	•
Lens Options		All SunSync® Products/ Polarized	All SunSync® Products/ Polarized	All SunSync® Products/ Polarized	Polarized	All SunSync® Products/ Polarize

**Patient Benefits:** Improved visual performance, as well as reduced eye stain, fatigue, neck, backm and shoulder pain when on a computer.

*Visual Need:* For progressive wearers who work on a computer for more than two hours per day or perform tasks requiring extensive use of intermediate vision.

Material	Power Range	Power Range Unity Via Wrap	Add Power Unity Via Designs	Add Power Unity Via OfficePro	Cylinder
CR-39	-7.00 tp +6.00	-7.00 to +6.00	+0.50 to +4.50	+1.00 to +3.50	Up to -6
Polycarbonate	-10.00 to +6.50	-6.50 to +6.00	+0.50 to +4.50	+1.00 to +3.50	Up to -6
Trivex	-10.00 to +6.00	-5.00 to +4.00	+0.50 to +4.50	+1.00 to +3.50	Up to -6
1.60	-10.00 to +6.00	-6.50 to +5.25	+0.50 to +4.50	+1.00 to +3.50	Up to -6
1.67	-12.00 to +8.25	-6.00 to +5.00	+0.50 to +4.50	+1.00 to +3.50	Up to -6
1.74	-15.00 to +10.00	-	+0.50 to +4.50	+1.00 to +3.50	Up to -6

UNITY RELIEVE							
Design	VSO® Code	CR-39	Polycarbonate	Trivex	1.60	1.67	1.74
unity Relieve 50	BA	TA	•	•	•	•	•
unity Relieve 70	BA	TA	•	•	•	•	•
Lens Options (not Recommended with TechShields® Blue)		All SunSync® Products/ Polarized	All SunSync® Products/ Polarized	All SunSync® Products/ Polarized	Polarized	All SunSync® Products/ Polarized	

**Patient Benefits:** Single vision lenses that reduce the effects of digital eye strain from device use. **Visual Need:** For single vision patients who use digital devices for more than two hours per day.

#### Reminders

• Select TA code when ordering Unity Relieve and charge the patient \$10.

UNITY SINGLE VISION							
Design	VSO® Code	CR-39	Polycarbonate	Trivex	1.60	1.67	1.74
SVx, SVxtra, SVxtreme	BA	•	•	•	•	•	•

Visual Need: Ideal for prescriptions exceeding +3.00 or -3.00 and/or greater thank 1.50 cylinder.

Material	VSO® Code	CR-39
CR-39	-12.25 to +6.00	Up to -6
Polycarbonate	-14.00 to +6.00	Up to -6
Trivex	-14.00 to +6.00	Up to -6
1.60	-14.00 to +6.00	Up to -6
1.67	-16.00 to +8.00	Up to -6
1.74	-16.00 to +10.00	Up to -6

<sup>\*</sup>The cylinder range is an estimated range that can be produced properly, but in terms of calculus, the range is eider. Listed are basic availability guidelines. Upper limits may vary based on frme size, decentration, and blank size.

PREFERRED ENHANCEME	NIS		
Light-Reactive Lenses	VSP Photochronic Category	VSP Code	Description
SunSync Elite (gray/brown0 SunSync Elite XT (gray)	Photochronics -Plastic	PR	<ul> <li>Ultra-fast reaction times</li> <li>UV Protective and targeted blue light friction</li> <li>Excellent indoor clarity and outdoor comfort</li> <li>Extra dark; retains color in the car (SunSync Elite XT</li> </ul>
SunSync (gray/brown0 SunSync Drive XT (gray)	Photochronics -Plastic	PR	<ul> <li>Fast reaction times</li> <li>UC protection and targeted blue light filtration</li> <li>Excellent indoor clarity and outdoor comfort</li> <li>Extra dark; excellent in-car color retention (SunSync Drive XT)</li> </ul>
Anti-Reflective Coatings	VSP AR Coating Category	VSP Code	Description
TechShield Elite (optional UV <sub>R</sub> )	D	QV (QV + BV)	<ul> <li>Superior visual performance; enhanced cosmetic appearance</li> <li>Improved scratch resistance, smudge resistance, cleanability</li> <li>Super oleophobic; super hydrophobic</li> <li>Optional backside UV protection</li> </ul>
TechShield Plus (optional UV <sub>R</sub> )	C	QT (QT + BV)	<ul> <li>Superior visual performance; enhanced cosmetic appearance</li> <li>Improved scratch resistance, smudge resistance, cleanability</li> <li>Oleophobic; hydrophobic</li> <li>Optional backside UV protection</li> </ul>
TechShield (optional UV <sub>R</sub> )	В	QN (QN + BV)	<ul> <li>Superior visual performance; enhanced cosmetic appearance</li> <li>Improved scratch resistance, smudge resistance, cleanability</li> <li>Hydrophobic</li> <li>Optional backside UV protection</li> </ul>
TechShield Blue	D	(QV + BV)	<ul> <li>Targeted blue light defense; combats digital eye strain</li> <li>Superior visual performance; enhanced cosmetic appearance</li> <li>Improved scratch resistance, smudge resistance, cleanability</li> <li>Super oleophobic; super hydrophobic</li> <li>Backside UV protection included</li> </ul>
UV <sub>p</sub>		BV	► Reduce back surface reflections

#### Reminders

- $\cdot \ \mathsf{Pair} \ \mathsf{all} \ \mathsf{SUnSync} \ \mathsf{products} \ \mathsf{with} \ \mathsf{TechShield} \ \mathsf{(not} \ \mathsf{TechShield} \ \mathsf{Blue)} \ \mathsf{for} \ \mathsf{optimal} \ \mathsf{light-reactive} \ \mathsf{performance}.$
- Charge additional \$10 for backside UV on Any TechShield AR Coating with UV<sub>R</sub>

	unity		techshield	sunsync <sup>-</sup>	
PRODUCT PORTFOLIO	DIGITAL FREEFORM PROGRESSIVE	DIGITAL FREEFORM SINGLE VISION	ANTI-REFLECTIVE COATINGS		LIGHT-REACTIVE LENSES
BEST	Unity Via Elite/Elite II VSP N	Unity Relieve (for digital strain relief) 50 or 70 Recommend pairing with TechShield Blue  VSP TA (\$10) + D	TechShield Blue  (for blue light reduction)  Backside UV included (add'l \$10 unless UV Covered  TechShield Elite Backside UV included (add'l \$10 unless UV Covered	UTRA FAST	SunSync Elite Dark to clear in seconds.  SunSync Elite XT Extra Dark. Ultra Fast.  VSP Category Photochromics - Plastic
BETTER	Unity Via Plus/Plus II Unity Via Wrap/Wrap II VSP O	Unity SVxtra	<b>TechShield Plus</b> Optional Backside UV (\$10)  VSP C	CONVENTIONAL	SunSync Drive TX Works behind the windshield SunSync VSP Category Photochromics - Plastic
GOOD	Unity Via/Via II VSP F	Unity SVx	Unity SVx  TechShield Plus  Optional Backside UV (\$10)  VSP B		
LIFESTYLE	Unity Via OfficePro (for computer/workplace/task) 5 ft. and 10 ft.  VSP IA  Unity Via Mobile/Mobile II (for hand-held digital devices)	Unity SVXtreme (for wrap frames)	Recommend solutions to		

#### **Notes**

- ▶ Position of wear measurement optional with O and N Category Unity Via Designs
- ▶ Simply take Monocular Distance PD and OC height for ALL Unity Designs
- ▶ \$10 copy for optional backside UV on TechShield Elite, TechShield Plus, TechShield
- ▶ SunSync products available in both Gray and Brown (SunSync Elite XT gray only)

#### **Guarantee**

- ▶ All Unity lenses backed by satisfactiob guarantee
- ► SunSync Light-Reactive Lenses one-year, 100% satisfaction guarantee (VSp members)
- ▶ TechShield AR Coatings two-year unlimited warranty


TECHNOLOGY	PATIENT BENEFIT	Unity Via Elite/Elite II	Unity Via Plus/Plus II	Unity Via/Via II
Advanced Fit	Allows the patient to easily find intermediate, near, and preferred reading distance.  Allows for more natural posture and unsurpassed visual ergonomics.	•		
InnoVue Technology	Provided wider and sharper visual fields by reducing peripheral blur and distortion.	•	•	
Digital Viewpoint	Optimized prescription at every point of the lens. Minimized peripheral distortion.	•	•	
OptiScreen Technology	Expanded near and intermediate-near areas (horizontally and verticallly) for easier viewing of digital displays in all configurations.	•	•	•
EquiBalance Technology	Better correspondence of binocular images that assures more comfortable and natural vision.	•	•	•
Automic Reading Height Optimization	Each lens is customized for the individual patient ensuring that the full add power is fit inside teh frame.	•	•	•
Variable Inset	Large usable reading area.	•	•	•

#### **BEST FITTING PRACTICES:**

#### **All Unity Lens Designs**

- ▶ Pre-adjust frame before measuring for optimal customization and performance
- ▶ Ensure patient's eyes are centered in the lens (even more important if the patient's distance RX is greater than =/-4.00).
- ▶ Take monocular distance PD and OCs.
- ▶ Provide lab actual OC measurement as taken; DO NOT drop from measured OC.
- ▶ Ideal position of wear fit parameters:
  - Back vertex distance ~14 mm by adjusting the nose pad
  - Pantoscopic tilt ~12°
  - Positive wrap angle (bends toward the face) of around ~5°
  - · Default measurements are used if none provided

#### **Unity Via OfficePro**

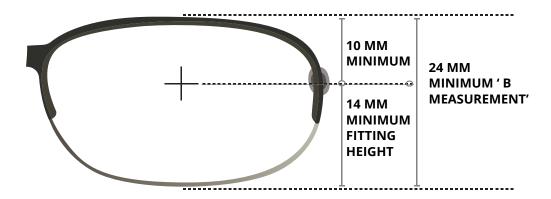

- Minimum OC height of 14mm.
- ▶ Vertical distance between OC and upper inside edge of frame should be at least 10mm.
- ▶ Once you receive the patient's Unity Via Office-Pro Lenses, remember to verify total near power in the reading area ONLY.



## **UNITY QUICK REFERENCES**

#### **Fitting Steps**

Adjust the frame before measuring.

Measure monocular PDs, OCs, and/or fitting heights.

Fit from the center oif the pupil to the deepest part of the lens.

Take Position of Wear (POW) measurements for a personalized visual solution (optional for category O and N lenses).

**Note:** Defaults are used when POW is not taken.

Default Measurements	Unity Via	Unity VIa OfficePro	PL Series
Pantoscopic Angle	12°	12°	9°
Vertex Distance	14mm	14mm	13mm
Frame Wrap	5°	5∘	5°
Frame Wrap (SVxtreme)	15⁰	-	15°
Minimum Fitting Height	14mm	14mm	12mm
Minimum "B"	-	24mm	-

#### **BEST PRACTICES:**

#### **Maximize Patient Experience**

- Discuss patient's visual needs for everyday, work, and play.
- Recommend solutions to address their needs.
- Explain how recommendations will provide them their best vision experience.

#### **Maximize Member Benefits**

- ▶ Verify VSP member coverage prior to patient visit.
- ▶ Maxmize VSP member benefits so patients receive the greatest value and the best vision experience possible (e.g., if proggressives are covered, offer N category).
- ▶ Apply Special member offers, multiple pair discounts, and/or rebates.

#### **Ordering Lenses**

- ▶ Pair Unity Lenses with Preferred Enhancements.
  - Choose SunSync Light-Reactive lenses for photochromic orders.
  - Choose TechShield AR Coatings for AR orders.
- ▶ Select Unity by name in the alphabetical drop-down menu when ordering.
- ▶ **Don't** select "Lab Choice" when ordering Unity products to ensure you receive the Unity lens you've recommended for your patient's best visual experience.

**NEED MORE INFO?** PLEASE CONTACT THE **RELEVANT VENDOR** 

Transitions

800-533-2081 www.transitions.com

877-414-7809 www.iotlenses.com

800-877-7195 www.unitylenses.com

shamir

877-514-8330

www.shamir.com

888-236-2219

www.neurolens.com

**ISSUE WITH AN ORDER? CALL US** 

(972) 488-3462



myedolab.com